


People with back pain can **STILL LIVE LIFE.**



Victory Medical
USA



THE V-FORCE FLEX

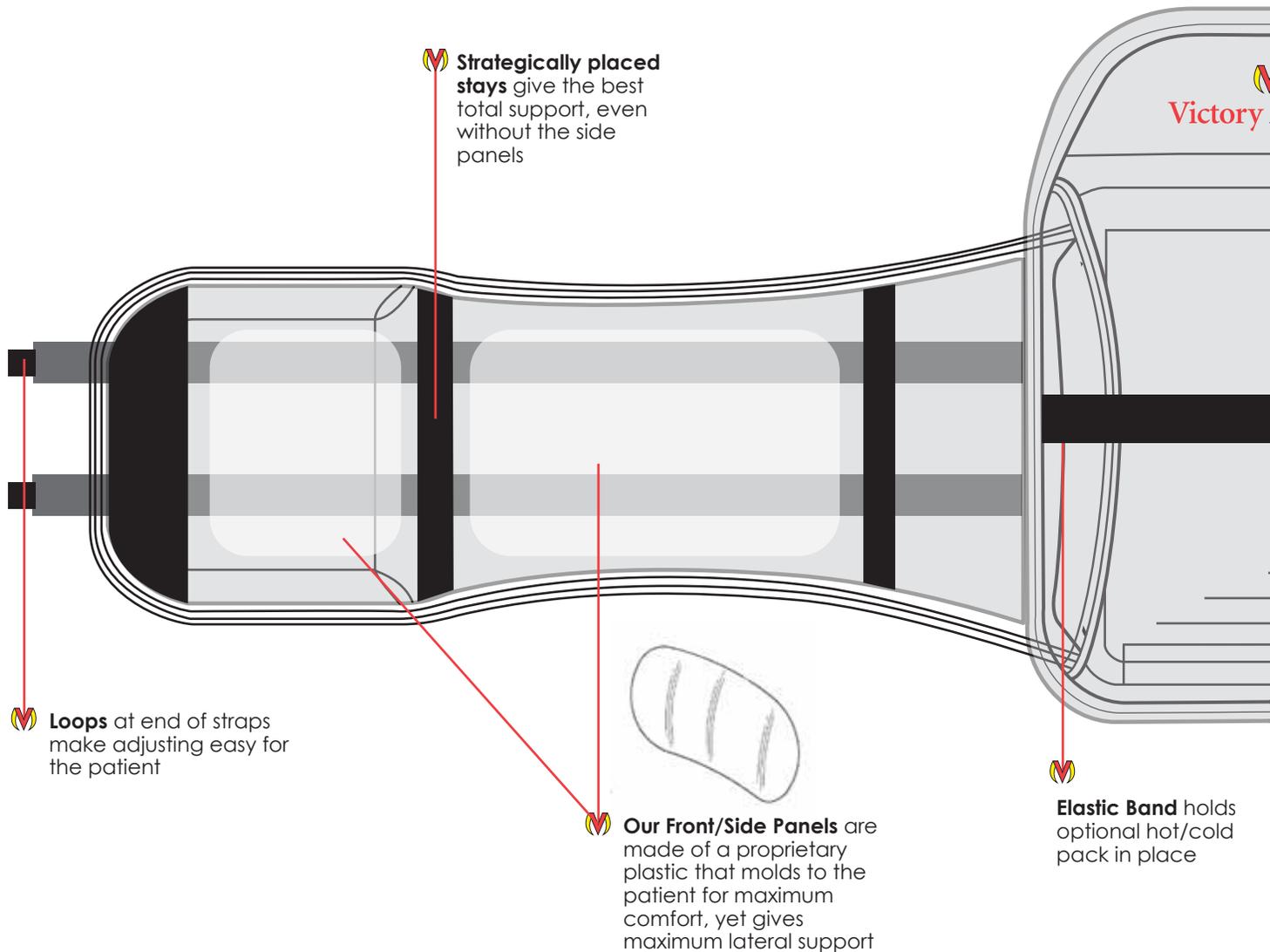

Strategically placed stays give the best total support, even without the side panels

Loops at end of straps make adjusting easy for the patient

Our Front/Side Panels are made of a proprietary plastic that molds to the patient for maximum comfort, yet gives maximum lateral support

Elastic Band holds optional hot/cold pack in place

V-FORCE

In physics, a force is any influence that causes a free body to undergo a change in speed, a change in direction, or a change in shape.

With its high-tech advanced design and extremely light weight, the V-Force Flex still has the ability to unload the low back. With its lateral support, the brace not only blocks unwanted motion, but it also creates hydrostatic lift, making it extremely comfortable.

INDICATIONS


The V-Force Flex back brace is indicated for the effective treatment of: Chronic low back pain, IDET's, post-op, herniated disc, spondylolisthesis, spondylolysis, facet syndrome, osteoarthritis, radiculopathy, degenerative disc syndrome, epidural injections.

V-FORCE OUTCOMES



Supreme fit, extreme compression and support, excellent range of sizes, great patient compliance and easy to don and doff. Because of all the overall features that have been put into the V-Force Flex brace, it enables the brace to function like no other, as far as blocking motion and

Medical
USA


Anatomically correct, molded back panel is designed with a built in, 15 degree angle, to fit perfectly into the lordic curve, working simultaneously with custom fit side panels to create circumferential lift

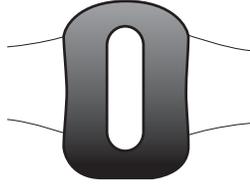

Our V-Foam Tech is proprietary fabric and padding, designed to breathe, keeping areas cool and comfortable for better patient compliance

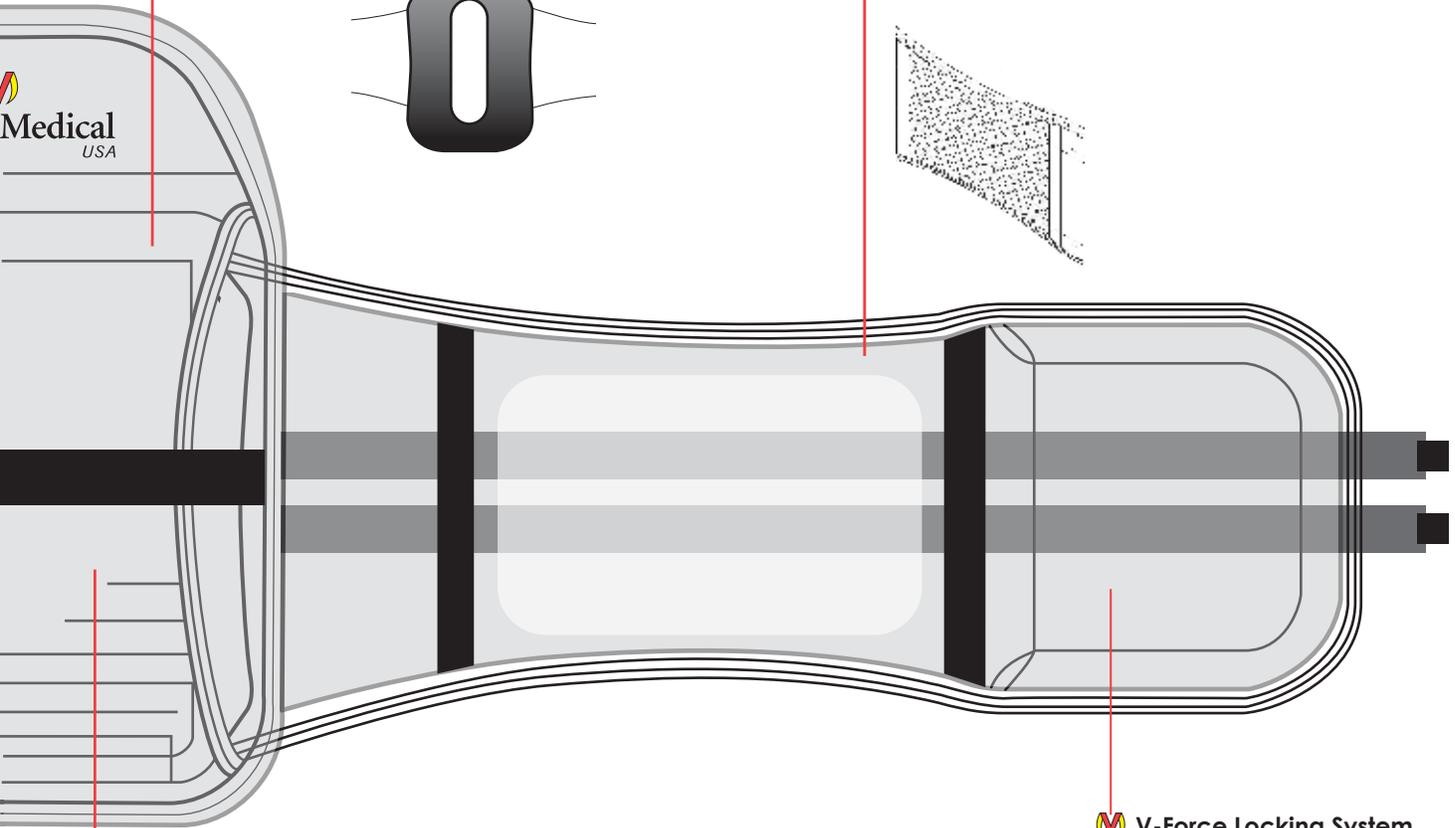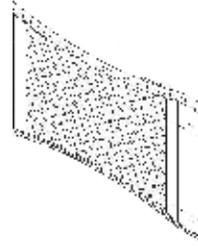

Unique pulley strapping system engages the **Vector Force Locking System** while pushing uniquely designed back panel perfectly into place creating a 5:1 mechanical advantage

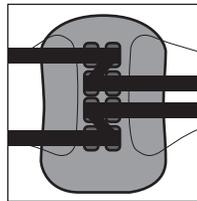


V-Force Locking System allows patient to control compression, therefore creating maximum control and confidence

MADE IN THE USA 


pelvic control. The shearer technologies of the brace adds confidence and stability for the patient.

The V-Force Flex creates circumferential lift and the uniquely designed back panel pushes the lordic curve into the proper position, which relieves pressure in the low back and creates space between the vertebrae, allowing the disc a chance to recede.

All panels are designed to be anatomically correct and can be installed and removed throughout the patients treatment program all making for better patient comfort and compliance.

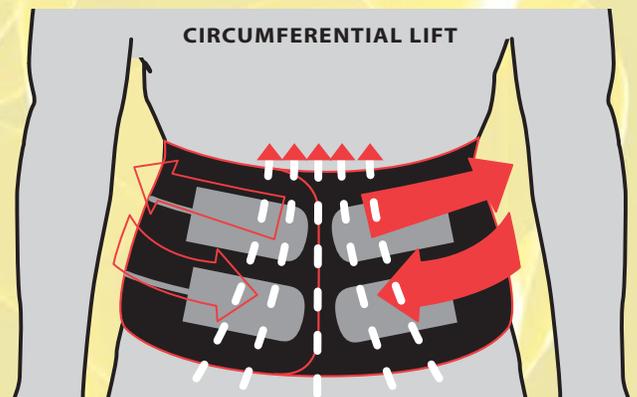

The illustration displays the way that space is created between vertebrae when the V-Force Flex is used. Our brace unloads the spine, creates a circumferential lift, and creates maximum support and comfort.

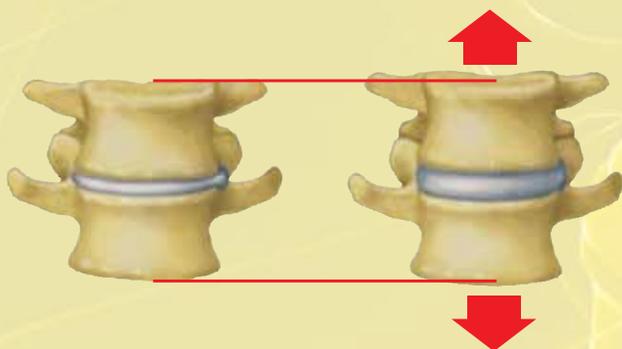

MODEL#	DESCRIPTION	APPROVED PDAC CODE
VFF2011	STANDARD BACK PIECE 15 1/2" H	LO637
VFFS2012	BACK PIECE 11 1/2" H <i> Special order for patients under 5'2"</i>	LO637
VFF2011A	STANDARD BACK PIECE 15 1/2" H WITHOUT SIDE PANELS	LO631
VFFS2012A	BACK PIECE 11 1/2" H WITHOUT SIDE PANELS <i> Special order for patients under 5'2"</i>	LO631

SIZING CHART	
<i>Measure around waist at navel</i>	
XXSM	22"-26"
XSM	25"-28"
SM	25"-31"
MEDIUM	31"-35"
ML.....	31"-35"
LARGE.....	35"-42"
X-LARGE	42"-46"
2XLARGE.....	45"-51"
3XLARGE.....	50"-56"
4XLARGE.....	55"-61"
5XLARGE.....	61"-66"



Victory Medical
USA

8501 BASH STREET, SUITE 1100
INDIANAPOLIS, IN 46250

PHONE: (317) 288-5122
www.victorymedicalusa.com